

MV-SAMPLER

Hematology sampler



Recommended for Vision Hema[®] blood cell image analysis system as well as for routine microscopy

- Standardized process for high-quality blood smear preparation
- Simultaneous preparation of 2 smears
- Setting of thickness and size of the smear
- Compact and lightweight
- Simple and easy to use

Quality blood smear in only 3 simple steps



1

Place a drop of blood on the target area



2

Depress the lever carefully but firmly and release it



3

Get a clean, thin film containing a large area

MV-SAMPLER

Specifications

Method of operation	manual
Length of the smear	adjustable
Number of slides proceeded simultaneously	2
Periodical adjustment	not required
Periodical lubrication	not required
Operation principle	mechanical
Power	not required
Operation temperature	15–35 °C
Operation humidity	25–80 %
Storage temperature	–50 – +55 °C
Storage humidity	25–93%
Size	150x90x140 mm (LxWxH)
Weight	0.4 kg

* Product images are shown for reference only and final product may differ

